



Symmetrikit Chair Cover – Care Sheet

Our 'suede style' covers are easy to remove and launder in a standard washing machine. The inner black cover should usually be wiped down but when necessary, can be removed and washed in a standard machine.

The chair frame should be wiped down with a damp cloth and mild detergent and left to completely dry before refitting the fabric covers. The frame can be steam cleaned or pressure washed if necessary. When cleaning powered chairs, please ensure the mains or battery supplies are disconnected before washing. Ensure the chair is thoroughly dry before reconnecting the power.

Washing details:

- **We recommend washing the Panache covers at 40°C with a non-biological detergent**
- **PU inner covers should be wiped clean with a damp cloth and mild detergent but can be washed to 60°C with a non-biological detergent**
- **Ward/Cuero chair covers can be wiped clean with a damp cloth and mild detergent**
- **DO NOT WASH ANY OF THE FOAM COMPONENTS**

All equipment supplied by Symmetrikit, which can be laundered, can be decontaminated in accordance with NHS guideline HSG (95)18 document.

“In a 40 degree Centigrade wash containing non biological washing powder”

The washing process should have a disinfection cycle in which the temperature of the load is maintained at 65°C for not less than 10 minutes or preferably 70°C for not less than 3 minutes. With both options “mixing time” must be added to ensure heat penetration and assured disinfection.

Waterproof covers can also be wiped using a 1% dilution of a chlorine based disinfectant.